

# OXYGEN BOTTLE HOLDER & CASE

FRESH AIR, EVERYWHERE



*movinglife*<sup>®</sup>

## SEATING & SUPPORT



ATTO /ATTO  
SPORT Armrests



Seat cushion



Seat cushion  
leather



Luxury Swivel  
Seat Leather



Seat & Cushion  
Support Belt



Luxury Orthopedic  
Backrest

## STORAGE



Essentials  
Pouch



Front & rear  
basket



Caryall  
& Cushion



Caryall  
& Cushion leather



Pet Carrier &  
Cushion Black



Flight Kit



Oxygen Bottle  
Holder & Case

## TECHNICAL



Cane / Crutches  
Holders



Mobile Phone  
Holder



Rear Light Lifting  
Handle



ATTO Left Hand  
Throttle Kit



Standard Flight  
Battery



Launch - H2 2023

Long distance  
battery - not flight  
approved



Cable kit


**ATTO Accessories** are created specifically to enhance, support and work with the ATTO mobility scooters in all user modes; while driving, while wheeling along as trolley and when folded for storage or transportation.

SCAN TO LEARN  
HOW TO COMBINE  
OUR ACCESSORIES




# How to Use

## OXYGEN BOTTLE HOLDER & CASE

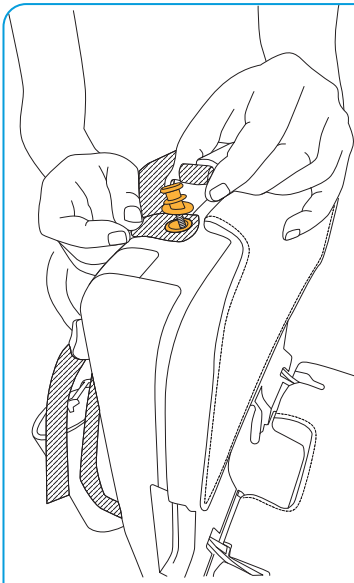


01

Unscrew the plastic knobs attached to the back seat. Insert the loop of the oxygen bottle holder rear section under the lift handle, and screw back the knobs

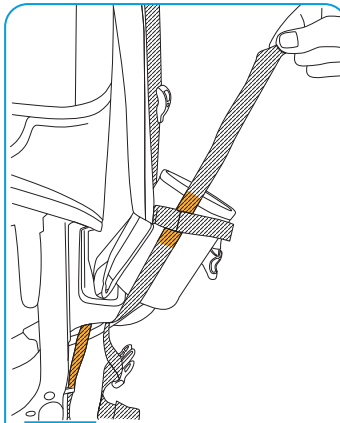


02

Pass the straps through the gap at the back of the chair and then through the straps on the oxygen bottle holder

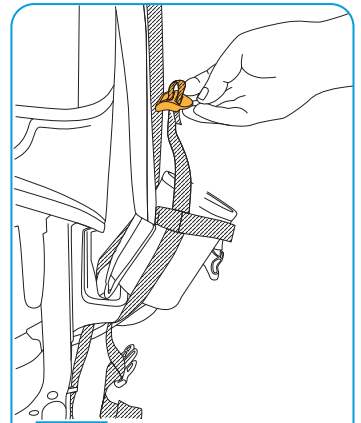


03

Connect the straps together by threading a strap through the loop

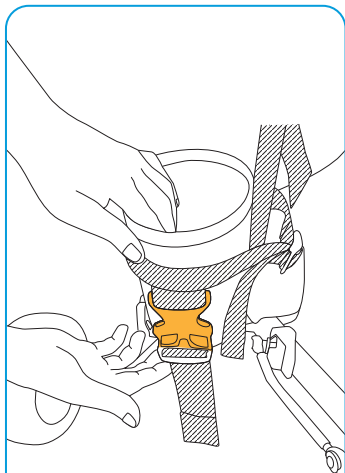


04

Tighten the rear buckle to secure the bottle holder

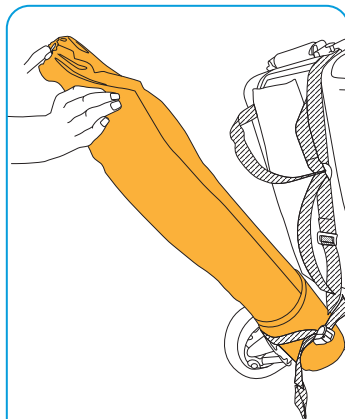


05

Insert the oxygen bottle into the bottom of the bottle holder

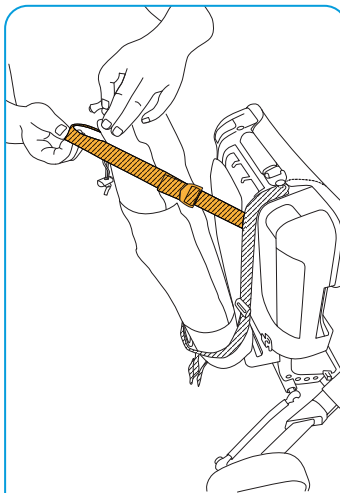

06

Fasten the strap to the top of the container

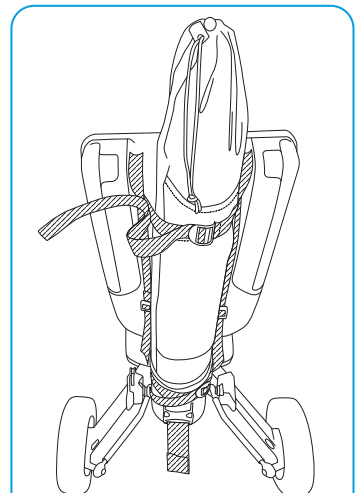

07

The product is ready to use.