Video Guides


English **6**

Deutsch **11**

Français **16**

Nederlands **21**

Dansk **26**

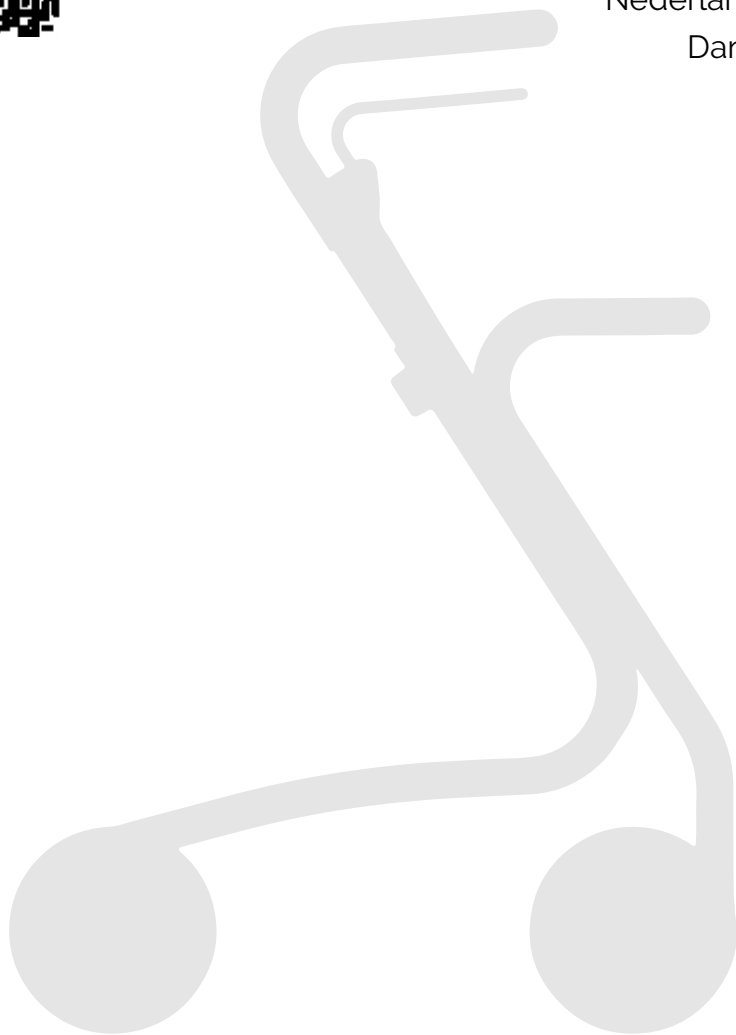Scandinavian Butler

by **ACRE**

Warranty/ Gewährleistung/ Garantie/ Garantie/ Garanti

EN Tested and approved according to EN ISO 11199-2:2005 and meets all the provisions of the MDR 2017/745 which apply to it. This product is a medical equipment (Class I).

Scandinavian Butler is covered by a 2-year warranty. The warranty does not cover moving parts, misuse and normal wear. byACRE offers lifetime warranty on the steel frame for the original owner. Simply register your product at www.byacre.com within three months after the purchase.

To keep the warranty please remember to save your proof of purchase.

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

Husk at gemme din købskvittering.

Attention/ Achtung/ Attention/ Let op/ Advarsel

1



EN Correct position of hands when walking. **DE** Richtige Position der Hände beim Gehen. **FR** Position correcte des mains pendant la marche. **NL** Correcte positie van de handen tijdens het lopen. **DA** Positioner hænderne korrekt på håndtaget ved brug.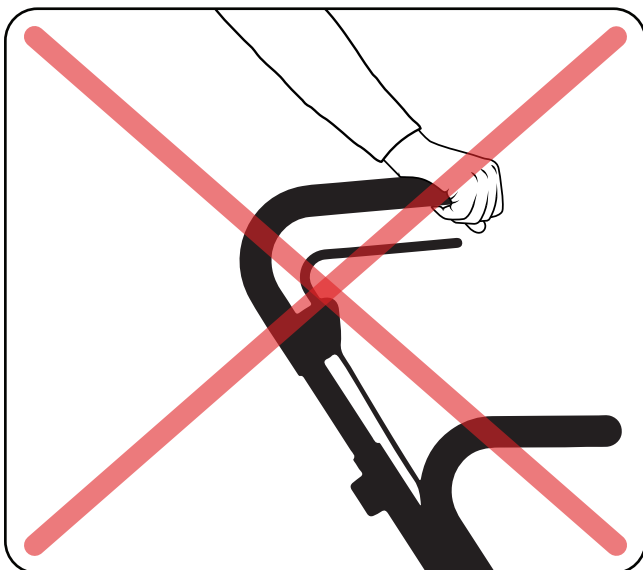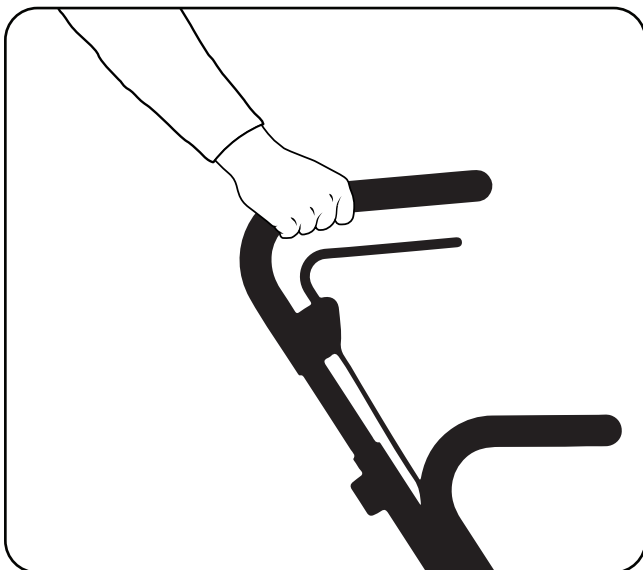

Attention/ Achtung/ Attention/ Let op/ Advarsel

2


EN Lift the front wheels when going over high obstacles. **DE** Heben Sie die Vorderräder an, wenn Sie über hohe Hindernisse fahren. **FR** Soulever les roues avant lors du franchissement d'obstacles élevés. **NL** Til de voorwielen op wanneer u over hoge obstakels rijdt. **DA** Løft rollatoren over forhindringer på gulvet.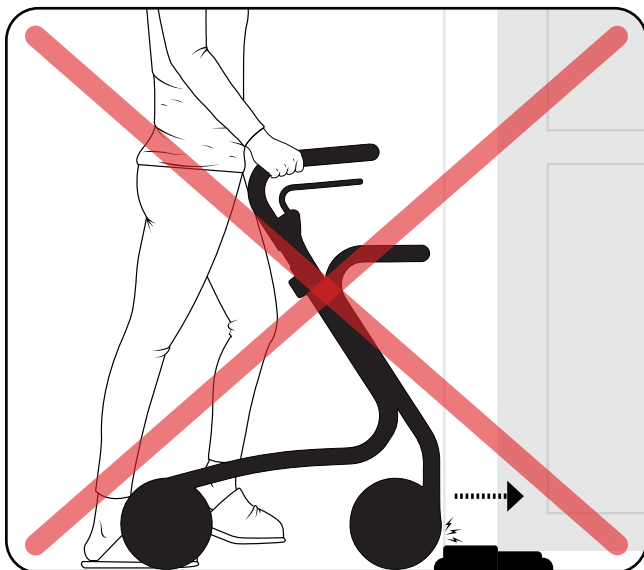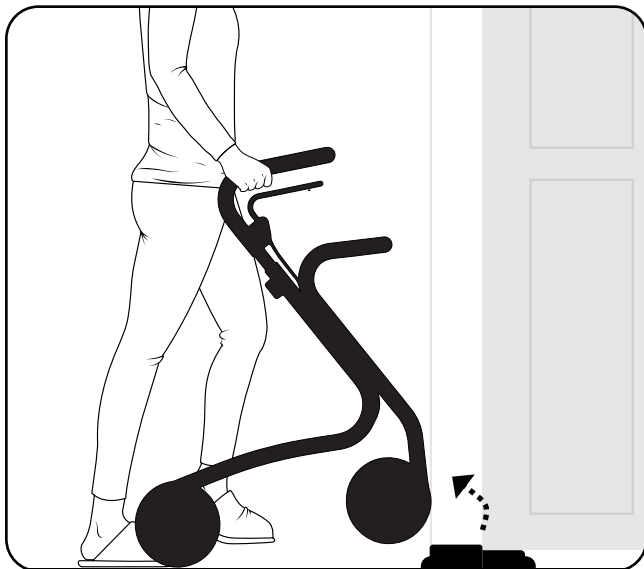

Attention/ Achtung/ Attention/ Let op/ Advarsel

3

EN Keep the rollator close to your body. **DE** Halten Sie den Rollator nah am Körper.
FR Garder le déambulateur près du corps. **NL** Houd de rollator dicht bij uw lichaam.
DA Undgå stor afstand til rollatoren.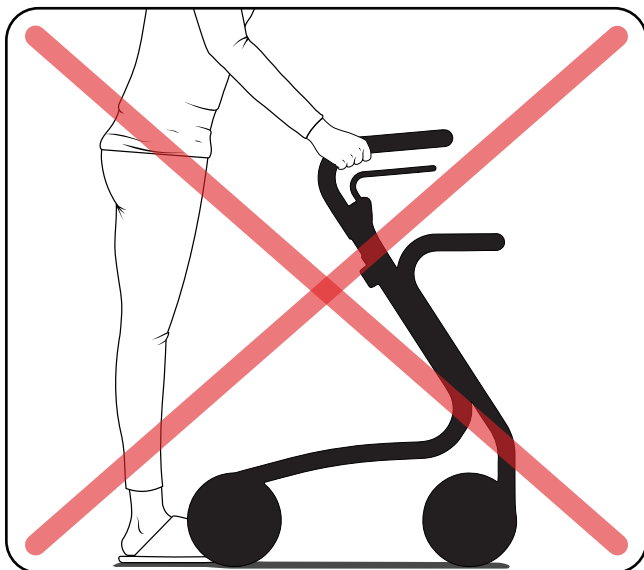


English

Congratulations on your new Scandinavian Butler. It is the world's most reliable rollator. In order to utilise all the possibilities the Scandinavian Butler provides, it is important that you carefully study this user manual.

Technical data is available towards the end of the user manual.


- ⚠ To avoid scratches during the assembly of the wheels, place your rollator on a soft surface.
- ⚠ When mounting the wheels make sure the locking features are engaged, normally by a clicking sound.

Attaching the rear wheels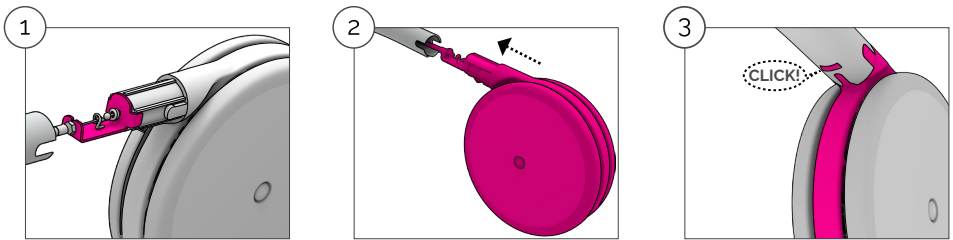





1. Make sure the metal bracket is positioned as shown here. When placed correctly, it should not move from side to side.
2. Push the wheel towards the frame while carefully guiding the brake cable into it to avoid damage.
3. Align the locking features of the rear wheel with the cut-outs on the edge of the frame tube and press it further into the frame until it clicks out through the slot on the underside of the frame.

Attaching the front wheels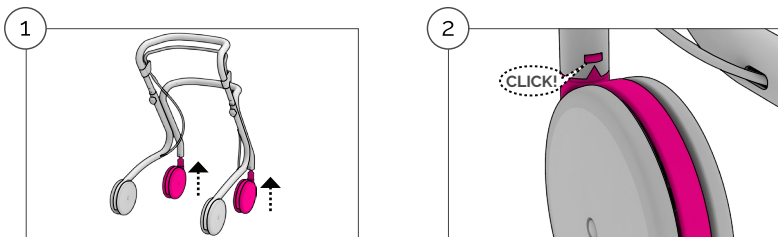

1. Insert the front wheel bushings into the frame.
2. Align the locking feature of the bushing with the cut-outs on the edge of the frame tube and press it further into the frame until the locking feature clicks out through the slot on the frame.



Height adjustment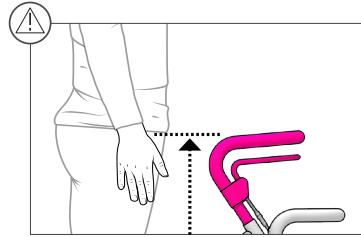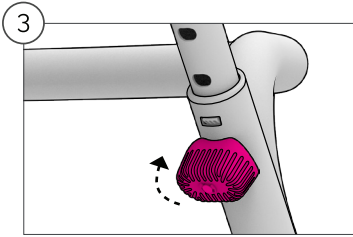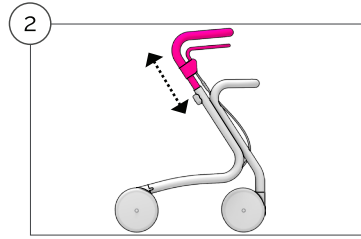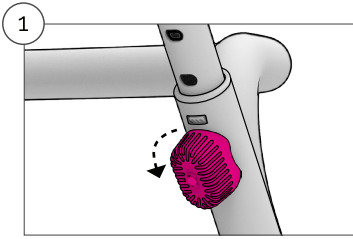

1. Turn both height adjustment knobs to 'open position', so the knobs are positioned across the frame tubes.
2. The handle can now be set to the desired height by pulling up and down with both hands. Note: the handle should be level with your wrists when standing in an upright position. If in doubt, consult your physical or occupational therapist.
3. Lock the handles by returning the knobs to the starting position (parallel to the frame tubes). Make sure the handle is locked in a secured position on both sides.

Brakes


Scandinavian Butler is equipped with a single brake lever that allows single hand maneuvering and braking. The brakes act on the rear wheels.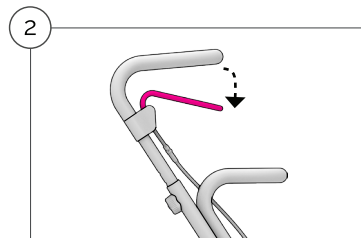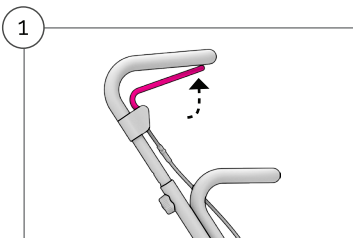

1. Pull the brake lever upwards to activate the brake. The lever moves back to the neutral position when the brake lever is released.
2. The parking brake is activated by pressing the brake lever down until you hear a click sound. Release the parking brake by pulling the brake lever back to the neutral position.

Adjusting the brakes

The brake should be adjusted if the brake force applied to the wheel is insufficient (this can occur if the tyres are worn) or the parking brake is difficult to activate (which is most often seen after replacement of a worn-down tyre). Always set the handle to the desired height before the brake is adjusted as the position may cause a little variation of the brake adjustment.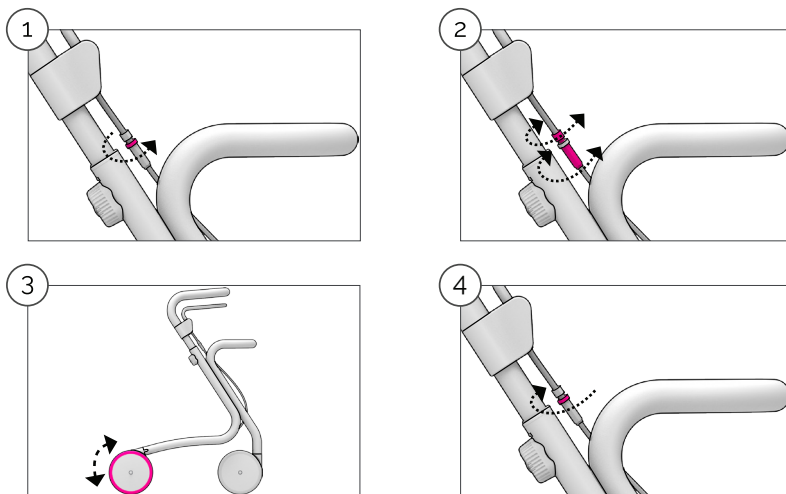

1. Loosen the safety nut.
2. Adjust the brake by holding either the adjusting screws or the adjusting nut. Unscrew it to loosen the brake or fasten it to tighten the brake.
3. Check that the wheels can rotate freely without any noise. Engage the parking brake to check if it works properly.
4. Tighten the safety nut.

Maintenance & cleaning

Scandinavian Butler should be regularly maintained by removing dust and dirt.

All surfaces can be cleaned with a textile cloth and clean water, if necessary, with a little non-abrasive detergent. Wipe with a dry cloth afterwards. Do not use paper towels as these can scratch the surfaces.

It may be necessary to occasionally tighten one or more screws with a H2,5 key.

Parts subject to wear such as wheels and brakes should be replaced in good time before they may pose a safety risk. Any defective part must be replaced immediately.

Safety precautions before use

- ⚠ Never use the Scandinavian Butler if it is in any way defective. Contact your dealer for servicing. In case of any serious injuries related to the rollator, please contact byACRE and/or your National Board of Health.
- ⚠ Before each use, check that the brakes are working properly.
- ⚠ The rollator is not a toy and should only be used as intended.
- ⚠ Check that the height adjustment of the handle is correctly engaged, and that the handlebar is not set higher than the maximum setting.
- ⚠ Do not change/modify the rollator. Such action could negatively impact the safety of using the rollator and expire the manufacturer's liability.
- ⚠ Never use non-original accessories with the rollator, as this could pose a potential risk for your safety.
- ⚠ Do not attach bags or other items to the handle. This can cause the rollator to fall.
- ⚠ If exposed to extreme temperatures, especially over prolonged periods, some parts of the rollator may feel hot/cold.

Safety instructions for use


- ⚠ Always position your hands correctly on the handlebar when using the rollator, as demonstrated in Section 1 on page 3. Incorrect hand positioning may cause the rollator to tilt forward.
- ⚠ Walking upright and looking ahead improves balance. Pay attention to obstacles on the floor. Lift the front wheels when going over high obstacles such as door thresholds.
- ⚠ For good support and safe walking, the rollator must be held close to the body. Pushing the rollator far in front of you creates a risk of falling.
- ⚠ Do not sit on the rollator nor use the rollator to drive a person around. The rollator is not designed to carry a person or heavy loads.
- ⚠ This rollator is intended for indoor use only.
- ⚠ Do not tilt the rollator more than max 5 degrees to the side. This could cause risk of falling!
- ⚠ Always use the brakes when stopping or parking. The brakes can be pulled up a little to decrease the speed while walking.
- ⚠ Note that the rolling speed of the rollator can be faster than your walking speed and create a risk of falling.
- ⚠ Do not use the rollator on stairs.
- ⚠ Always make sure that your clothes are not trapped or stuck in the handles or in other parts of the rollator.

Accessories

Tray & Basket set

- Includes:
- Tray & basket holder in metal
 - Versatile flat tray that allows you to carry or move objects around the house
 - Soft and roomy basket to store your daily necessities

Installation guide

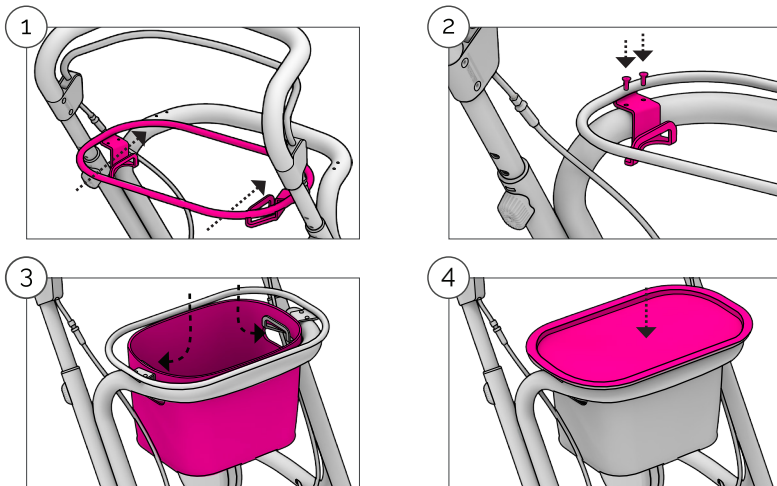1. Grab the tray & basket holder and carefully push it towards the frame with both hands. Be careful not to scratch the frame. Tip: Have someone hold the rollator in place or place the rollator against a wall.
2. Tighten both sides with the rivets provided.
3. Place the felt basket onto the brackets of the holder.
4. Fit the tray onto the holder.

- ⚠ Regularly check that the holder is properly attached.
- ⚠ Always make sure that both sides of the basket are hooked onto the brackets of the holder.
- ⚠ Always make sure the tray is properly placed and supported by the holder.

Technical specification

	Tray & Basket holder	Tray	Basket
Material	Carbon steel	Birch wood and cork	100% Polyester (felt)
Height	55 mm (2.1")	15 mm (0.6")	215 mm (8.4")
Width	415 mm (16.3")	420 mm (16.5")	Bottom: 270 mm (10.5") Top: 330 mm (13.0")
Depth	260 mm (10.2")	260 mm (10.2")	175 mm (6.8")
Weight	870 g (1.9 lbs)	370 g (0.8 lbs)	200 g (0.4 lbs)
Load capacity	7,5 kg (16.5 lbs)	5 kg (11 lbs)	2,5 kg (5.5 lbs)

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