Video Guides

Get started! **2**


English **5**

Deutsch **8**

Français **12**

Nederlands **15**

Dansk **19**

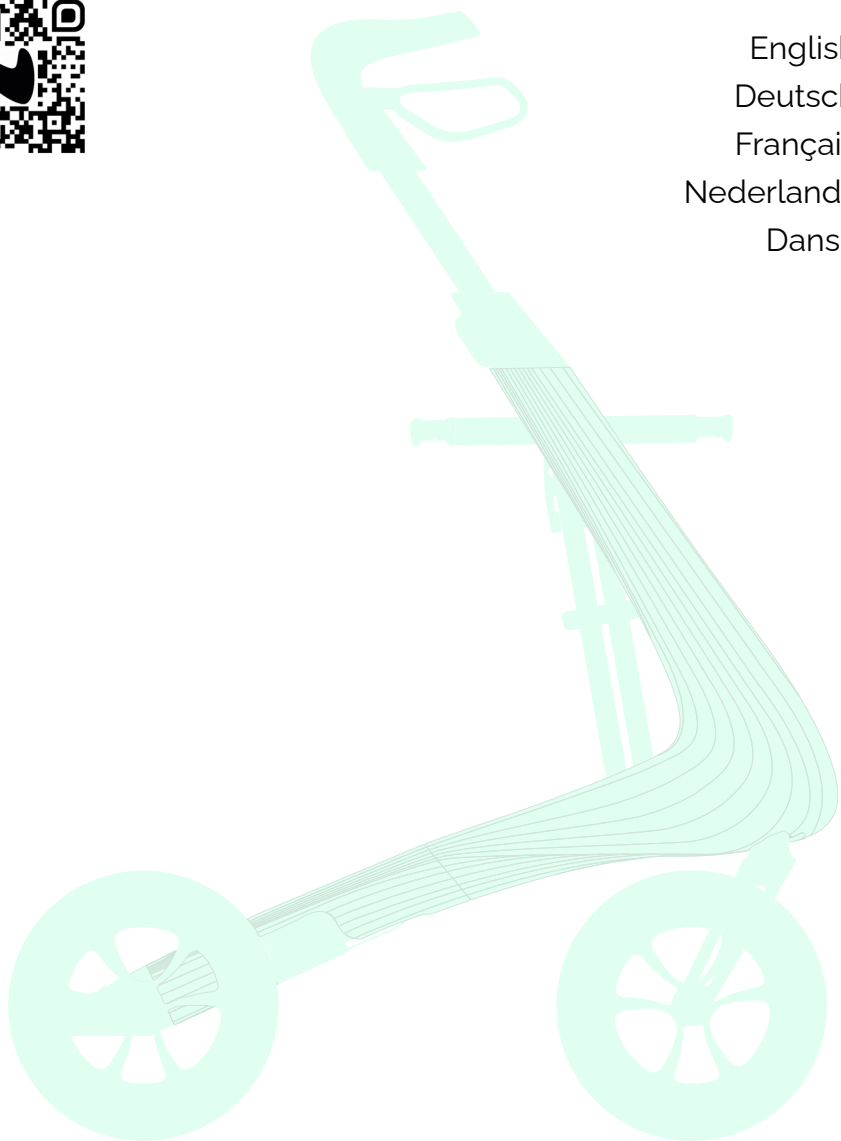
Nordic Pioneer Rollator

by  **ACRE**

GET STARTED!

Unfolding/ Auseinanderfalten/ Ouverture du rollator/ Uitvouwen/ Udfoldning

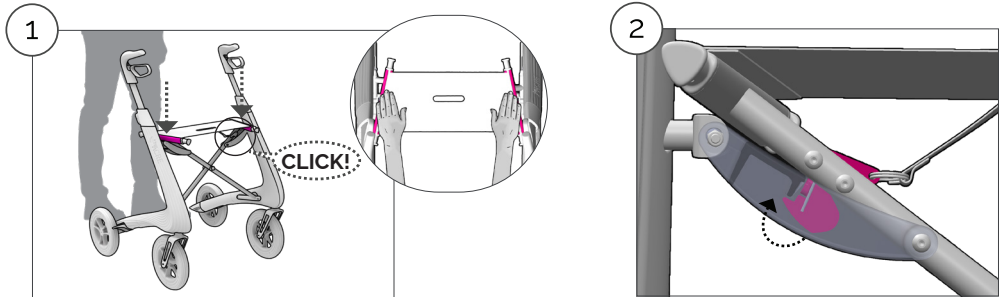- ⚠ **EN** Push the seat rails down until you hear a click. Check that the hook is locked in place.
- ⚠ **DE** Den Sitz nach unten drücken bis ein hörbarer Klick erfolgt. Überprüfen Sie ob der Verriegelungshaken richtig eingerastet ist.
- ⚠ **NL** Duw naar Beneden aan beide uiteinden van het zitje totdat u én klik hoort. Gelieve na te kijken dat het haakje onder het zitje goed is vastgeklikt.
- ⚠ **FR** Poussez vers le bas de l'assise jusqu'à ce que vous entendiez un click. Veuillez vérifier que le crochet d'accrochage en dessous du siège soit bien fixé.
- ⚠ **DA** Pres ned på sædets yderramme, indtil du hører et tydeligt "klik". Tjek, at låsekrogen under sædet er i korrekt indgreb.

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Warranty/ Gewährleistung/ Garantie/ Garantie/ Garanti

EN Tested and approved according to EN ISO 11199-2:2005 and meets all the provisions of the MDR 2017/745 which apply to it. This product is a medical equipment (Class I).

Nordic Pioneer is covered by a 2 year warranty. The warranty does not cover moving parts, misuse and normal wear.

byACRE offers lifetime warranty on the aluminium frame for the original owner. Simply register your product at www.byacre.com within three months after the purchase.

To keep the warranty please remember to save your proof of purchase.

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Husk at gemme din købskvittering.


English

Congratulations with your new Nordic Pioneer rollator. It is the world's most tempting rollator. In order to utilise all the possibilities the Nordic Pioneer provides, it is important that you carefully study this user manual.

Technical specifications and overview of accessories are available toward the end of the user manual.

Brakes

Nordic Pioneer is equipped with two independent brakes acting on the rear wheels.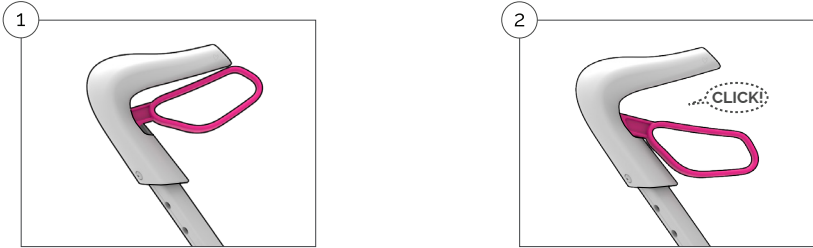


1. Pull the brake levers upwards to activate the brake. The lever moves back to the neutral position when the brake lever is released.
2. The parking brake is activated by pressing the brake lever down until you hear a click sound. Release the parking brake by pulling the brake lever back to the neutral position.

- ⚠ Always engage both parking brakes when you sit on the seat.
- ⚠ Under normal circumstances both brakes should be activated simultaneously.
- ⚠ Be aware that when only one brake is activated, Nordic Pioneer will turn to the same side as the engaged brake.
- ⚠ Do not store with the parking brake activated as prolonged activation of the parking brake can leave visible marks on the wheel and decrease the braking performance.

Adjusting the brakes


The brake should be adjusted if the brake force applied to the wheel is insufficient (this can occur if the tyres are worn) or the parking brake is difficult to activate (which is most often seen after replacement of a worn down tyre). Always set the handle to the desired height before the brake is adjusted as the position may cause a little variation of the brake adjustment.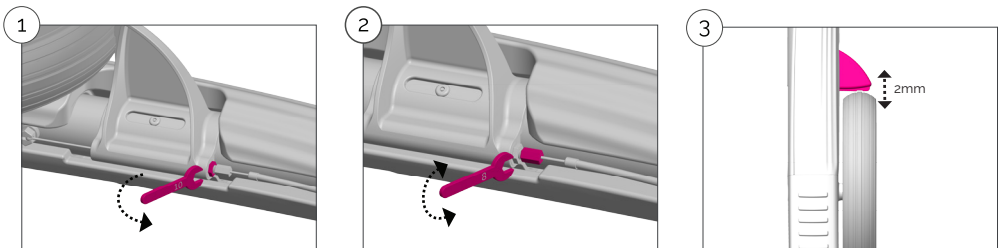

1. Loosen the safety nut with a 10mm wrench.
2. Adjust the brake by tightening/loosening the adjustment screw with a 8mm wrench. Fasten it to loosen the brakes or unscrew it to tighten the brakes.
3. Check the distance between the brake and the wheel. It should be 2mm. Then tighten the safety nut.

Curb lifter

Nordic Pioneer is equipped with a curb lifter that makes it easier to get over smaller obstacles like curbs and low steps. This is particularly helpful if heavy objects are transported in a front-mounted basket or bag.

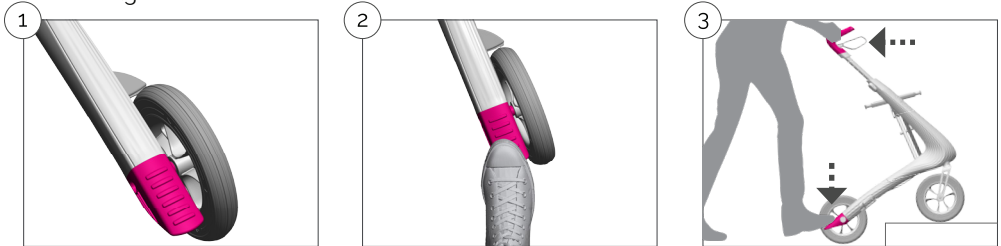- 1-3. Carefully press on the curb lifter with a foot. Pull slightly by the handles to lift the front wheels off the ground.

⚠ Be aware that the stability of Nordic Pioneer changes when it is reclined. Be cautious and try to bypass rather than climb high obstacles.

Accessories

Nordic Pioneer can be made even more useful with original accessories. A full display of our range can be found at www.byacre.com.

All of our accessories are model specific and of course tested with it to ensure your safety.

The accessories should always be installed according to the accessories manual. Maximum carrying capacity for bags are always specified in the manual.

⚠ Never use non-original accessories with the rollator, as this could pose a potential risk for your safety.

Maintenance & cleaning

Nordic Pioneer should be regularly maintained by removing dust and dirt. Pay special attention to keeping the brakes and wheel hub clean.

All surfaces can be cleaned with a textile cloth and clean water, if necessary with a little non-abrasive detergent. Wipe with a dry cloth afterwards. Do not use paper towels as these can scratch the surfaces.

Check the screws and nuts on a regular basis. It may be necessary to occasionally tighten one or more screws with a T25 / 10 mm wrench.

Parts subject to wear such as wheels and brakes should be replaced in good time before they may pose a safety risk. Any defective part must be replaced immediately.

We recommend that the seat is regularly impregnated. The seat can be cleaned with a wet cloth and, if necessary, with a little stain remover.

Safety precautions before use

- ⚠ Never use the Nordic Pioneer if it is in any way defective. Contact your dealer for servicing. In case of any serious injuries related to the rollator, please contact byACRE and/or your National Board of Health.
- ⚠ Attention: When unfolding the rollator, please check that the rollator is securely locked in place with an audible click.
- ⚠ Before each use, check that the brakes are working properly and the screws and nuts are tight.
- ⚠ The rollator must not be used by people weighing over 100kg/220 lbs. It is not suitable for transporting heavy loads or people.
- ⚠ The rollator is not a toy.
- ⚠ Check that the height adjustment of the handles are correctly engaged and that the handle bars are not set higher than the marked maximum setting.
- ⚠ Do not change/modify the rollator. Such action could negatively impact the safety of using the rollator and expire the manufacturer's liability.

Safety instructions for use

- ⚠ Attention: Always activate both parking brakes before sitting down and while seated. Do not use the rollator to drive a person around and do not move the rollator while seated. The rollator is not designed to carry a person while in motion.
- ⚠ For your own safety, you should not stand on the Nordic Pioneer, nor lean back, sideways or forward while seated.
- ⚠ Do not tilt the rollator more than max 5 degrees to the side. This could cause risk of falling!
- ⚠ Do not attach bags or other items to the handles. This can cause the rollator to fall.
- ⚠ Always use the brakes when stopping or parking. The brakes can be pulled up a little to decrease the speed while walking.
- ⚠ For good support and safe walking, the rollator must be held close to the body.
- ⚠ Walking upright and looking ahead improves balance. Particular care should be taken when going down curbs.
- ⚠ Do not push the rollator far in front of you as this creates a risk of falling.
- ⚠ Note that the rolling speed of the rollator can be faster than your walking speed and create a risk of falling.
- ⚠ Do not use the rollator on stairs.
- ⚠ Always make sure that your clothes are not trapped or stuck in the handles or in other parts of the rollator.
- ⚠ While riding on public transport, sitting on the rollator is not safe.

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