

rollz[®]

takes you further



Ergonomically and stylishly designed rollators

Living life to the fullest, that is what Rollz is all about. Be it for a walk in the park, a visit to a museum, a holiday trip or a shopping afternoon. Our ergonomically and stylishly designed rollators will bring you further, while also supporting an active and independent life.



**"Now I dare to walk longer with
the rollator because I know I can
rest in the wheelchair on the way
if I get tired."**




rollz motion

Enjoy adventures with Rollz Motion²

The 2-in-1 rollator and wheelchair

The Rollz Motion² is perfect for those who want to continue their journey but worry about how far they can go. This 2-in-1 mobility equipment allows you to walk in comfort and also benefit from the ability to transform the rollator into a wheelchair when needed.

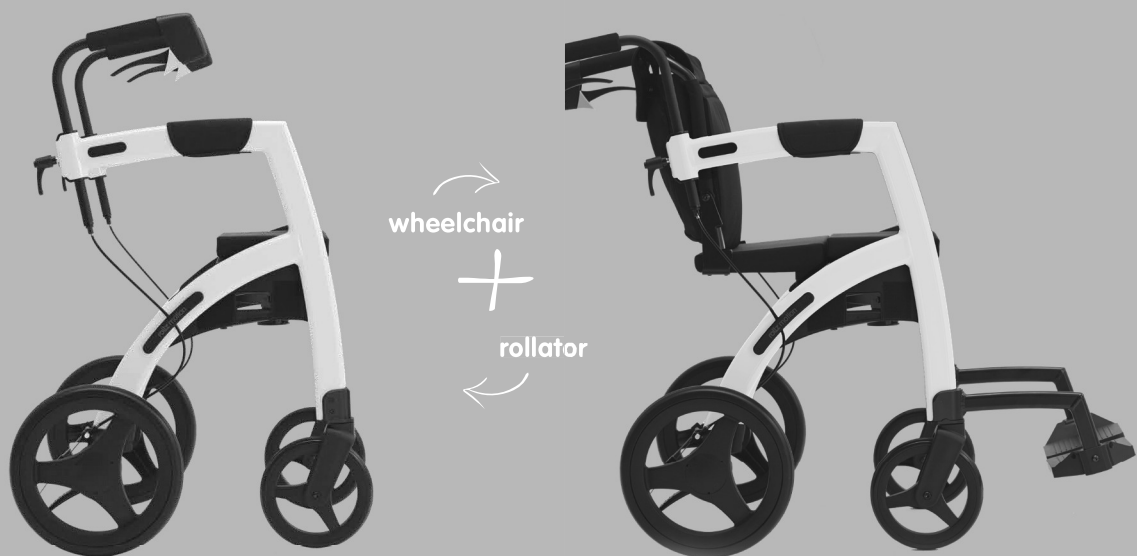

See the Rollz Motion in action:




Easy to transform

With a flick of the wrist you can transform the Rollz Motion² rollator into a wheelchair and back again. This will give you confidence to go further and rest along the way, while enjoying a day out with your family or friends.

The **Rollz Motion²** is available in four colours: Pebble white, Dark purple, Matt black and Island blue.

The **Rollz Motion² Small** is available in Island blue or Matt black.

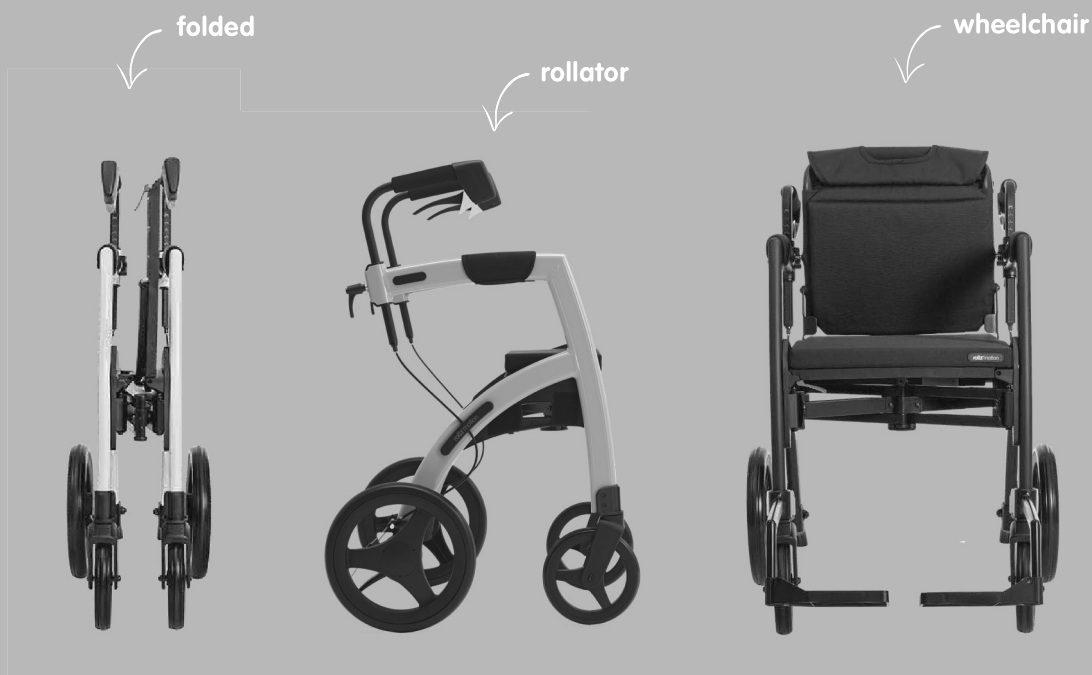

rollz motion

Product features

Rollz Motion²

Stylish

Award winning Dutch design rollator and wheelchair in one, for going out in style.

Safe

Innovative drum brakes, suitable to wet and dry conditions.

Convenient

Rest on a comfortable cushioned seat or transform the rollator into a wheelchair.

Easy aid

Cross doorsteps and kerbs easily with the kerb assistant.

Stable

Walk tall and steadily between the ergonomic hand grips that are adjustable in height.

Compact

The Rollz Motion² is easily foldable into a compact package that fits in almost any car.

Comfortable

The large rear wheels with soft tyres and foam handles ensure a comfortable walk.

Manoeuvrable

Light steering and easy motion with the manoeuvrable front wheels.





rollz motion
electric

Rollator and electric wheelchair combo

The rollator, transport chair & electric wheelchair in one

The Rollz Motion Electric is a rollator, transport chair & electric wheelchair in one. This versatile 3-in-1 tool keeps the user active and independent regardless of mobility.



See Rollz Motion Electric
in action:



Independence at a higher level

The Rollz Motion Electric combines the features of three devices, helping people with mobility limitations to stay as independent as possible. The rollator option offers stable support while walking, allowing users to make the best of their mobility. When walking becomes difficult, the user can attach the wheelchair package and rest or continue moving while safely seated. The electric system further enhances the functionality by making it a motorized wheelchair, enabling users to travel independently at any time.

- Switches seamlessly between the three mobility-aid options
- Turns on the spot even in narrow spaces
- Easy foldability that allows compact transportation
- Comfortable pneumatic tires



rollz motion
electric

Walk or drive (yourself)

Adaptable

Durable and practical solution for multiple mobility needs, from requiring some support to wheelchair use.



Push support

Features an automated pushing power that provides electric push support to ease pushing uphill on ramps, slopes, and hills.



Joystick

Equipped with joystick controller that allows for independent use of the wheelchair.



Modular

The electric system can be attached to any Rollz Motion version, providing users with the freedom to travel independently.

NEW | Future
of
mobility



rolz motion
rhythm

Parkinson rollator with adjustable cues

The Rollz Motion Rhythm is developed for people with a disrupted gait pattern or who suffer from freezing episodes

This very stable rollator encourages the start of a rhythmic gait pattern with the help of three cues: laser, sound and vibration.

See the Rollz Motion Rhythm in action:



Regain your walking pace with Rollz Motion Rhythm

This versatile rollator can be used by physiotherapists and other professionals supporting patients rehabilitating from neurological or mobility-impacting injuries.

Three Cues

This Parkinson rollator gives multiple stimuli to step out of a freeze: a laser line on the ground, vibration in the handlebars and a metronome with two different sound tones. The three cues can be combined as desired.



rollz motion
rhythm



Start the pattern

The laser, vibration and sound indicate the desired walking rhythm. You choose the cues and set the tempo in the module on the rollator. By pressing the button on the handlebar, the rhythmic pattern starts.




Easy to set and activate

CUES | see
feel
hear


Speed up or slow down

The pace of the cues can be lightly accelerated to stimulate the user to walk faster. Or the pace can be set slower, for smoother movements.

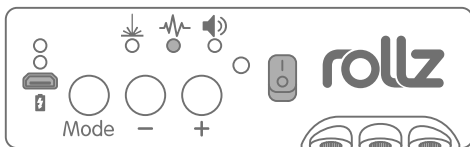


Two tones

Variation in pitch of the metronome prevents habituation, this way the user stays focused. These tones can also be played via the smartphone.

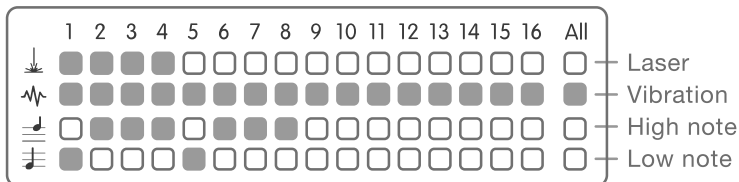

Smartphone app

The vibration, pitch and sound intensity can be adjusted in the corresponding app on the smartphone. Via the app, the pattern length of the cues can be set to: 8x, 16x, 32x or nonstop.

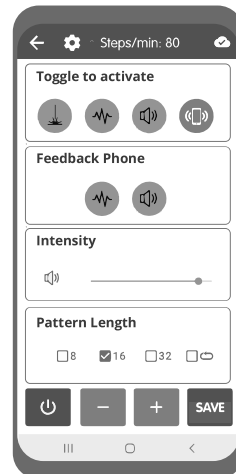

Best New
Product
2022



rollz motion
performance

Smooth and safe ride on uneven terrain

All terrain rollator and wheelchair in one

Thanks to the four pneumatic tyres, the Rollz Motion Performance provides a comfortable ride, even on bumpy roads. The extra thick padding of the armrests and the ergonomically shaped handles add to the comfort. Moreover, the footrests are adjustable in height.



Product features

Rollz Motion Performance

Stylish 2 in 1

Dutch design for going out in style, which transforms from rollator to wheelchair within seconds.

Ergonomic grips

Ergonomically shaped soft handles for optimal grip.

Eye-catching colour

Modern matt coating in the new colour Jungle green.

Visible

Visible in the dark thanks to multiple reflective parts.

Soft padding

Armrests with extra thick foam padding to increase comfort.

Firm seat

Wedge-shaped cushion for a better sitting position.

Air filled tyres

Sturdy pneumatic tyres enable more comfortable rides on bumpy roads.

Stable

Walk tall and steadily, while safely supported by a sturdy design.



rollz motion
performance

Specifications

	Rollz Motion ²	Rollz Motion ² Small	Rollz Motion Performance	Rollz Motion Rhythm	Rollz Motion Electric
Rollator weight	11,6 kg	11,6 kg	11,4 kg	11,6 kg	22,6 kg (excl. wheelchair)
Wheelchair weight	3,4 kg	3,4 kg	3,4 kg	3,4 kg	3,4 kg
Maximum weight capacity	125 kg	125 kg	125 kg	125 kg	100 kg
User height	1,60 - 1,90 m	1,50 - 1,85 m	1,60 - 1,90 m	1,60 - 1,90 m	1,60 - 1,90 m
Seat height	55 cm	50 cm	55 cm	55 cm	55 cm
Handle height range	87-98 cm	82-94 cm	86-98 cm	87-98 cm	88-99 cm
Seat size rollator	20 x 45 cm	20 x 45 cm	20 x 45 cm	20 x 45 cm	20 x 45 cm
Seat size wheelchair	42 x 45 cm	42 x 45 cm	42 x 45 cm	42 x 45 cm	42 x 45 cm
Overall width	67 cm	67 cm	67 cm	67 cm	74 cm
Overall length	63 cm	63 cm	63 cm	63 cm	71 cm
Width folded	31 cm	31 cm	29 cm	31 cm	44 cm
Front wheel diameter	8 inch	8 inch	8 x 1¼ inch	8 inch	8 inch
Rear wheel diameter	12 inch	12 inch	12½ x 2¼ inch	12 inch	12½ x 2¼ inch

WARRANTY: 5 years on the frame and 2 years on other parts with exception of normal wear and tear.

"I was looking for a rollator that does not look like one. Going shopping just feels good now as this rollator is so easy to manoeuvre!"



rollz flex

On the go with Rollz Flex²

Design rollator with large bag

At first glance, it does not appear to be a walking aid. This was exactly the intention of the designers, to create an enjoyable design for going outdoors. This rollator manoeuvres easily and you can walk stable and comfortable, while the bag carries your groceries.



See the Rollz Flex²
in action:




Manoeuvrable

You can easily manoeuvre anywhere, even in between obstacles. The front and rear wheels are set close to each other, and this way your rollator rotates close around you. It allows you to turn smoothly in a narrow space.

If you make a turn, the wheels do not get in the way. You can step aside easily and take something out of a closet or rack. The Rollz Flex² functions the same even when the bag is loaded to its capacity.

With ease you can go up the kerbs with the help of the kerb assistant.

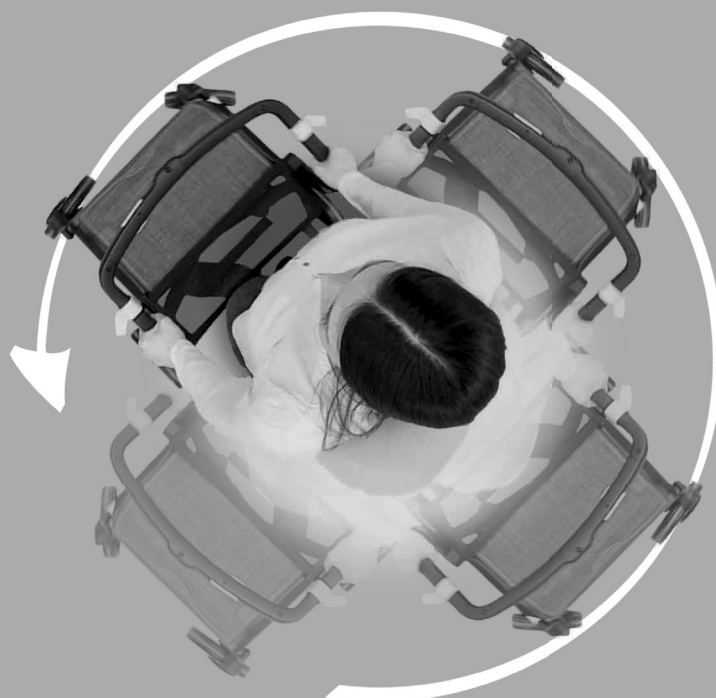

rollz flex

Product features

Rollz Flex²

Stylish

Lightweight rollator with the look of a designer shopper.

Flexible

Suitable for taller or shorter users, thanks to the easily adjustable push handle.

Practical

The large water-resistant bag can carry up to 20kg/25litre. Includes removable inner bag.

Comfortable

Rollz Flex² allows a comfortable ride with less vibrations thanks to the softer foam-filled tyres.

Manoeuvrable

Easy to push and steer thanks to its small turning circle.

Convenient

Rest on the leather look seat while having the handlebar as a supportive backrest.

Easy aid

Go over doorsteps and kerbs easily with the kerb assistant, even with a full loaded bag.

Light and compact

The Rollz Flex² weighs only 7.7 kg and folds easily and compactly.





prijs
favoriet

altijd goed
altijd goedkoop

de 10 beste van 1000 producten

CHIPS
natuurlijk

rollz flex

Specifications

	Rollz Flex²	Rollz Flex² Small
Rollator weight	7,7 kg	6,8 kg
User height	1,55-1,95m	1,40-1,60m
Seat height	62 cm	52,5 cm
Seat width	44 cm	44 cm
Weight capacity	125 kg	125 kg
Overall width	62,3 cm	62,3 cm
Overall length	60 cm	60 cm
Depth folded	33 cm	33 cm
Height	80-100 cm	69-81 cm
Bag capacity	25 liter/20 kg	14 liter/20 kg
Wheel diameter	8 inch	8 inch

The **Rollz Flex²** is available in four frame colours: Matt black, Pebble white, Dark purple and Jungle green. The bag is made from sustainable and water-resistant material and is available in grey.

WARRANTY: 5 years on the frame and 2 years on other parts with exception of normal wear and tear.





Rollz Motion travel cover

Due to the unique design of the travel cover, both the Rollz Motion and the wheelchair package are incorporated into one compact package. Using the travel cover will help preventing damages.



Rollz Motion shopper

This stylish and functional bag can be clicked onto the frame of the Rollz Motion. You can put your purchases directly into the shopper and take them with you, so there is no need to carry a shopping bag.



Rollz Motion cup holder

The Rollz cup holder is very easy to use. Thanks to the sliding system, the holder can be placed into the desired position quickly, enabling the user to take a water bottle along.



Rollz Motion 3-in-1 wheelchair package holder

With these holders you can also hang your wheelchair package over the seat. This gives you more walking space and freedom of movement. A cane and shopping bag holder are integrated.



Rollz Motion and Rollz Flex umbrellas

This umbrella protects you from rain and sunshine. Attach it to the frame so you can use both hands on the rollator for best support.

Rollz Motion back support

Out on your own and left the wheelchair package at home? This back support offers you a short rest, while leaning against the soft strap.



Rollz Motion seat belt

The seat belt keeps you fixed in the wheelchair for extra safety. It can be easily placed on the wheelchair package and adjusted to the desired length.



Rollz Flex cane holder

The cane holder attaches to the handlebar and stays there. The corresponding part has been designed to fit almost any walking stick.



Rollz Flex shopping hooks

If you go shopping, this set of hooks offers the possibility to carry along another shopping bag.



Rollz Motion and Rollz Flex tray

With this bamboo tray you can easily take a cup of coffee or a book with you everywhere you go indoors.



Rollz International BV
Rotterdamseweg 402M
2629HH Delft
The Netherlands

020 362 2010
www.rollzmobility.co.uk



[rollzint](#)



[RollzInternational](#)



[rollzint](#)



[RollzInternational](#)