Lifecare

Dear customer, thank you for the trust placed in EMO product. Choosing our line Antiescaras cushions are the most effective Lifecare you have acquired a range of quality products and high health status. Read these instructions carefully. If in doubt contact your doctor or specialist dealer.

Therapeutic Purpose Decubitus cushions are designed to be used by persons subject to long periods of sitting position or who have disabilities - for example, users of wheelchairs. Its main purpose is to prevent pressure sores in the superficial tissues; create comfort and help maintain posture absorbing impacts, while a suitable temperature is maintained in the contact zone. Its wide contact surface achieves adequate distribution of pressure, relieving surface areas and bony prominences of shock loads.

Furthermore, the model decubitus cushion independent air cells, is designed to SCL. Its flexible design contains cells connected together in two separate sections that manage pressure distribution and prevent decrease, facilitating the circulation of blood in the capillaries. This design facilitates the rectification of imbalances in the pelvis postural. Gracias conferring stability and control the properties of the sheath including the product, this is the hand for incontinent patients, decreasing the effects of skin maceration and facilitating transpiration.

Pillows Viscoelastic Lifecare are indicated as adjunctive treatment method of the cervical spine. Also for prophylaxis of cervical disc orders. Dan support for neck and shoulder muscles, improve quality of sleep, relieve stress and promote sleep. Pillows techniques are able to reduce the pressure on the nerve roots at the cervical level. They adapt to user needs for its ergonomic and comfortable design, achieving optimal pressure distribution, plus an anatomical neck and relaxed during the hours of sleep posture. They consist of a foam core with a strong viscoelastic character, capable of behaving in a unique way each user, and a sheath of comfortable textile touch, breathable and washable quick release velcro.

Presentations Lifecare decubitus cushions and pillows come in various models, differentiated based on the composition of the interior material of the product and its sheath according to their composition, shape and external dimensions.

MODEL: Decubitus Cushion Solid Silicone Gel. COVER: FIREPROOF M1. DIMENSIONS: 40X40X3 cm. PROFILE / FORM: 3 cm SQUARE.

MODEL: Silicone Gel Cushion Decubitus Solid and Foam. COVER: FIREPROOF M1. DIMENSIONS: 40X40X6 cm. PROFILE / FORM: 6 cm SQUARE and HORSESHOE.

MODEL: Decubitus Cushion Solid Silicone Gel and Foam Viscoelastica formed. COVER: hydrophobic and IGNIFUGA: FLAME RETARDANT. DIMENSIONS: 40X40X6 cm. PROFILE / Shape: 6 cm SQUARE and HORSESHOE.

MODEL: Cushion decubitus independent air cells. COVER: hydrophobic and IGNIFUGA: FLAME RETARDANT (M1) type or flame (in case of the models with memory foam cells and air), providing safety to the user with limited mobility. Furthermore, the case of viscoelastic material cushions and air cells is also hydrophobic which facilitates cleaning during use. Should be particularly careful in the Solid and Silicone Foam and Liquid Foam, avoiding snags, punctures or mechanical accident can cause leakage of the filling material inside the pad, making it unusable.

MODEL: Decubitus of Freely adjustable viscoelastic which facilitates cleaning during use. Should be particularly careful in the Solid and Silicone Foam and Liquid Foam, avoiding snags, punctures or mechanical accident can cause leakage of the filling material inside the pad, making it unusable. The material used in the manufacture of these products are biocompatible, nontoxic and irritable skin contact.

MODEL: Liquid Flotation Cushion decubitus. COVER: FIREPROOF M1.

en

DIMENSIONS: 40X40X6 cm and 42x42x6 cm. PROFILE / Shape: 6 cm SQUARE and HORSESHOE.

MODEL: Shaping Cushion Viscoelastic memory. COVER: hydrophobic and IGNIFUGA: FLAME RETARDANT. DIMENSIONS: 40X40X6 cm and 42x42x6 cm. PROFILE / Shape: 6 cm SQUARE and HORSESHOE.

MODEL: Basic Cushion Viscoelastic memory. COVER: hydrophobic and IGNIFUGA: FLAME RETARDANT. DIMENSIONS: 40X40X6 cm. PROFILE / Shape: 6 cm SQUARE AND HORSESHOE.

MODEL: Viscoelastic Pillow Lifecare. COVER: Cover fabric, breathable and washable quick release velcro. DIMENSIONS: 48 X 30 (09.07) cm. 67 X 35 X (8-10) cm.

Instructions for use

The pad should be placed on the seat and under the buttocks of the user, leaving the top base at the hand for incontinent patients. The side handles facilitate transport and handling of it. Make sure the pad dimensions are appropriate for the chair of destination, and do not wrinkle the pad, which could disturb the user are displayed. Regarding the viscoelastic pillows Lifecare, the outer cover is removable and hand washing recommended in warm water (maximum 30 °) and neutral soap. During washing, the Velcro should be glued together with a strong adhesive. Dry the pad in a dry nature. Do not store or iron the garment. Contact with ointments, creams, alcohols and solvents may damage the garment. Do not use solvents or any chemical that could impair the properties of the material. For added durability use with regular bed linen is recommended.

The Air decubitus cushion presents one / two (depending on the model) inflation valves and includes an air pump for filling and repair kit. To adjust the level of air cushion requires the user once inflated the cushion to sit on him keeping closed / s valve / s and open the valve letting out of the air inside to reach maximum stability. For the cushion air cells also endeavor (with the advice of the Technical Specialist) by opening and closing valves with the inflated cushion and the user above him, that between the prominence marrow lower and floor pad there 1.5cm and no less. At that time definitely will close the valves since they have acquired the right balance between efficiency, postural control and stability. In general, the amount of air must be allowed to escape depends on the weight of the person and its morphology. It should be noted that the more air have greater decubitus effect; and the less air stability. Is advisable to adjust the amount of air within the cushion 10 days.

We recommend at least every two hours, different beats getting up in the saddle pad to vary the support points and no sores encourage repetition. In the air cell cushions is not recommended weight greater than 130Kg. Pillows Viscoelastic Lifecare should be placed during times of rest at the anatomical cervical curve, keeping the correct position by positioning the side of the pillow with more height in the cervical area. The pillows are able to adapt to the anatomical needs of each user. They are suitable for all types of single beds pillows.

Use Precautions and Flammability The LIFECARE products are accompanied by a breathable outer jacket manufactured from washable fabric to 40. In addition, this outer sheath has the property of being fireproof retardant (M1) type or flame (in case of the models with memory foam cells and air), providing safety to the user with limited mobility. Furthermore, the case of viscoelastic material cushions and air cells is also hydrophobic which facilitates cleaning during use. Should be particularly careful in the Solid and Silicone Foam and Liquid Foam, avoiding snags, punctures or mechanical accident can cause leakage of the filling material inside the pad, making it unusable.

Avoid ingestion and contact with eyes or mucous membranes. It should be noted that the pad can resist transpiration and ventilation of the contact area, particularly noticeable in the Silicone Gel Solid, Solid Silicone Gel-Foam and Silicone Gel Solid-Visco, increasing local sweat. Do not bring the product to flame or heat source at elevated temperatures. The cushions of air cells are supplied with a repair kit if accidentally during use, the structure is punctured. Your repair is simple: the pad is positioned the adhesive patch the leak produced.

Keep pads horizontaly and boxed in a dry place. Regarding the viscoelastic pillows Lifecare if an allergic reaction should occur remove the product and consult with the prescriber or the retailer where it was purchased. Do not expose the garment to a flame or source of heat at high temperatures. Store at room temperature, preferably in the original packaging. Guarantee within the framework of laws.

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Quality Certificates All materials used have been tested, comply with applicable European standards and are of high quality and offer maximum safety and comfort during use. Öko-Text Standard 100: textile eco-label awarded after the completion of a catalog of criteria analysis to guarantee the absence of substances harmful to health. Certifies the absence of harmful substances in all materials that is made a pledge.

Warranty ORTHOPÉDIC MEDICAL SPECIALTIES, S.L. states that antiescaras cushions and pillows Lifecare comply with the applicable essential requirements in the regulations RD 1591/2009 for Health Products. We guarantee decubitus cushions and pillows Lifecare against defects in workmanship or material used for this, as stated in RD 1/2007. If it detects a defect within the warranty period, kindly contact your dealer to solve your problem as soon as possible.

The warranty is void if the product has been damaged by improper use, or have not followed the instructions and recommendations for use that are explained in this booklet.

IMPORTANT! Please do not forget to keep the receipt of purchase that will Guarantee (with date of purchase and sealed by the seller) and present the complete product (cover and body of the product), these requirements that are essential to any claim.

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