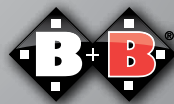
Bedienungsanleitung | User manual |  
Mode d'emploi | Manual de instrucciones

**Bremshebelverlängerung**  
**Brake lever extension**  
**Alargador de palanca de freno**  
**Extension du levier de frein**

*Enjoy mobility.*

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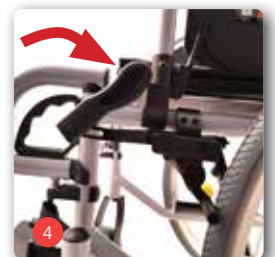
## 1. Preface

Dear users,

For most manual wheelchairs from B + B, a brake lever extension is optionally available as an original accessory. The extended brake lever reduces the actuation force of the brake and is therefore particularly suitable for people with reduced hand strength. This user manual serves as a supplement to the user manual for the main product and does not replace this.

Please follow this user manual when operating a B + B brake with brake lever extension.

Installation of the brake lever extension should be completed by your specialist dealer. Installation instructions are made available to specialist dealers.

If you have any questions, you can contact the B+B customer service department by telephoning +49 (0)700/6000 7070.

Important! Indicates particularly useful information in the specific context.

Caution! Indicates information that is specifically relevant to safety. Follow the operating manual!

### Note!

No liability is accepted for printing errors, product and colour modifications, or changes to technical specifications. The colours of the product shown may vary slightly in reality. The figures may contain accessories. The current version of each instruction manual is ready for download at the B+B website:

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## 2. Actuation of the parking brake using the brake lever extension

If your wheelchair is equipped with a brake lever extension, please refer to these instructions for use.

### 2.1. Brakes

- Before starting a journey, make sure that the brake levers are in the operating position (Fig. 1). The plastic handles must be pushed securely into the levers (Fig. 1 and 2).
- Bring the wheelchair to a standstill (see user manual for your main product).
- Push both left and right brake levers forward (Fig. 2).
- The wheelchair is now held safely in place.
- To release the brake, pull the levers back (Fig. 1).

### 2.2. Fold in the brake lever extension

- The brake lever can be folded in while driving or when changing sides.
- Pull the plastic handle out of the lever (Fig. 3).
- Fold the plastic handle over (Fig. 4).
- Before use, the brake must be returned to the operating position (Fig. 1) in the reverse order.

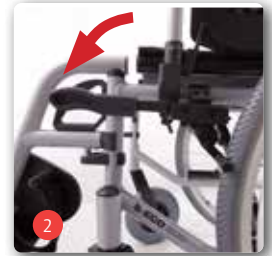
The parking brake only affects one driving wheel and must therefore always be applied on both sides!

Please observe the information on brake checks in the operating instructions for the main product.

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### **B+B France S.A.R.L.**
### **Bischoff & Bischoff GmbH**