HAND BIKE
CICLO A TRE RUOTE DA CORSA


Shark RS/RT

DIRECTIONS FOR USE
NOTICE D'UTILISATION
MANUALE D'USO
GEBRUIKERSHANDLEIDING
MANUAL DE USUARIO
INSTRUÇÕES PARA UTILIZAÇÃO

000690679.02

**QUICKIE®**

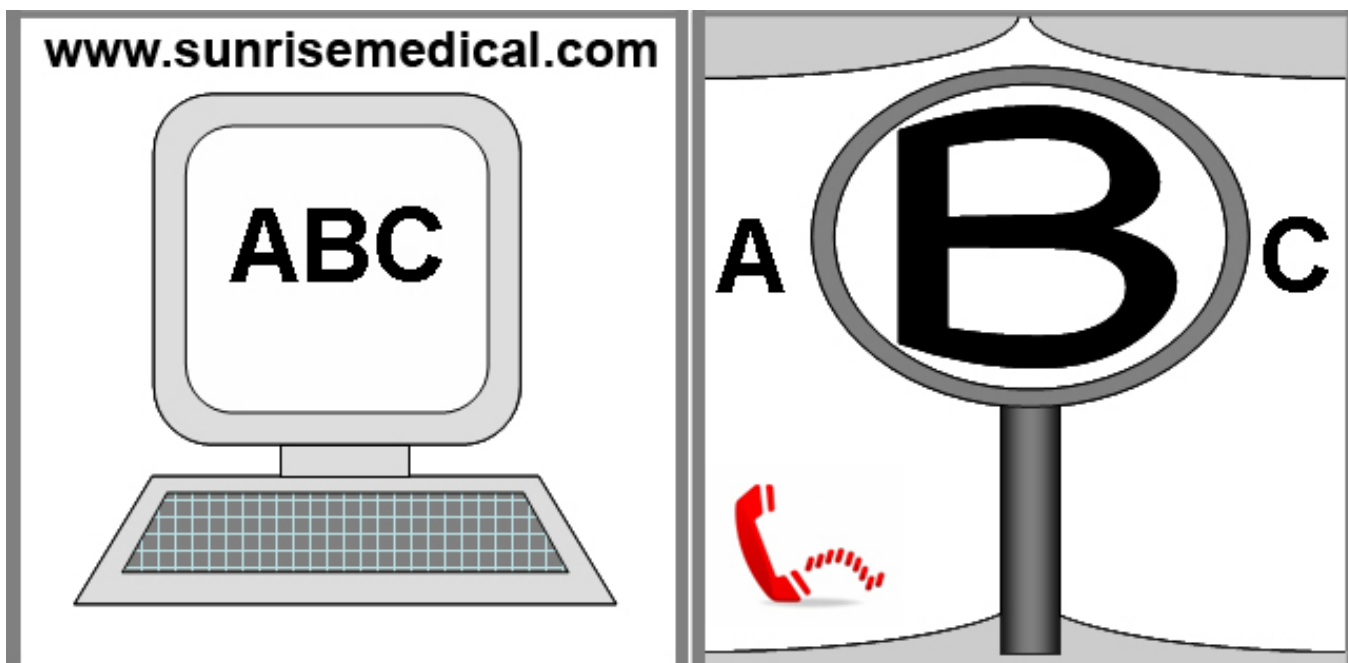Hand bike components

EN

SUNRISE MEDICAL is certified to ISO 9001 which ensures the quality of our products at all stages of development and production of this hand bike. This product meets all the requirements in accordance with EU directives. The options or accessories shown here are available at extra charge.

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HAND BIKE:

1. REAR WHEEL
2. HEADREST
3. BACKREST UPHOLSTERY
4. GEAR-CHANGE
5. FRONT WHEEL
6. LEG SUPPORT
7. PARKING BRAKE
8. IMPACT PROTECTION

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Foreword

Dear Customer,

We are very happy that you have decided in favour of a high-quality product from SUNRISE MEDICAL.

This owner's manual will provide numerous tips and ideas so that your new hand bike can become a trustworthy and reliable partner in your life.

Maintaining close ties to our customers means a lot to us at Sunrise Medical. We like to keep you up-to-date about new and current developments at our company. Keeping close to our customers means: fast service, as little red tape as possible, working alongside our customers. When you need replacement parts or accessories, or if you just have a question about your hand bike – we are there for you.

We want you to be satisfied with our products and service. At Sunrise Medical we are constantly working to develop our products further. For this reason, changes can occur in our palette of products with regard to form, technology and equipment. Consequently, no claims can be construed from the data or pictures contained in this user's manual.

The management system of SUNRISE MEDICAL is certified to EN ISO 9001, ISO 13485 and ISO 14001.

As the manufacturer, SUNRISE MEDICAL, declares that the hand bikes conform to the 93/42/EEC directive.

Please contact your local, authorised SUNRISE MEDICAL dealer if you have any questions concerning the use, maintenance, or safety of your hand bike.

If there is no authorised dealer in your area, or you have any questions, contact Sunrise Medical either in writing or by telephone.

**Sunrise Medical Limited
High Street, Wollaston
West Midlands, DY8 4PS
England
Tel.: +44 (0) 1384 446688
www.sunrisemedical.com**

IMPORTANT:

DO NOT USE YOUR HAND BIKE UNTIL THIS MANUAL HAS BEEN READ AND UNDERSTOOD.

Area of Application

The Shark RS / RT serves both as a physical training aid as well as an aid to help you integrate/take part in daily life.

The Shark RS / RT should enable the user to be able to safely cover long stretches both ergonomically and in an ecologically efficient manner.

The Shark RS /RT should be able to cope with tarmac surfaces and firm paths with different topography.

You must not exceed a speed which has been adjusted to meet safety requirements and take into account the capabilities of the user.

It is imperative that you carry out product maintenance and inspect the bike before every use

LIFETIME

The expected life of the hand bike is 5 years, provided that:

It is used in strict accordance with the intended use. All service and maintenance requirements are fulfilled.

Table of Contents

Hand bike components 3
 Foreword 4
 Area of Application. 4
 Definitions 5

1.0 Safety and hazard notes 6
2.0 Warranty 7
3.0 Handling 8

4.0 Options 8
 Brakes 8 - 9
 Footrest 10
 RT gearshift 10
 RS gearshift 10
 Backrest adjustment 11
 Height / depth adjustment of the fork 11
 Rear axle 12

5.0 Maintenance and Care 12
6.0 Tyre maintenance 13
7.0 Disposal / Recycling of Materials 13
8.0 Nameplate 14
9.0 Torque 15

Definitions

Definitions of words used in this manual

Word	Definition
 DANGER!	Advice to the user of potential risk of serious injury or death if the advice is not followed
 WARNING!	Advice to the user of a potential risk of injury if the advice is not followed
 CAUTION!	Advice to user that potential damage to equipment may occur if the advice is not followed
NOTE:	General advice or best practice
	Reference to additional documentation

NOTE:
 Please keep a note of your local service agent's address and telephone number in the space provided.
 In the event of a breakdown, contact them and try to give all relevant details so they can help you quickly.

Dealer signature and stamp

1.0 Safety and hazard notes

The technology and construction of this hand bike have been designed to provide maximum safety. International safety standards currently in force have either been fulfilled or exceeded in parts.

Nevertheless, users may put themselves at risk by improperly using their hand bikes. For your own safety, the following rules must be strictly observed.

Unprofessional or erroneous changes or adjustments increase the risk of accident. As a hand bike user, you are also part of the daily traffic on streets and pavements, just like anyone else. We would like to remind you that you are therefore also subject to any and all traffic laws. Be careful during your first ride in this hand bike. Get to know your bike.

Before each use, the following should be checked:

- Fixed axles or quick-release axles on the rear wheels.
- Velcro on seats and backrests
- Tyres, tyre pressure and wheel locks and main brake.
- Before each journey carry out a visual inspection of the frame, checking for cracks and other damage.

Before changing any of the adjustments of this hand bike, it is important to read the corresponding section of the Owner's manual.

NEVER exceed the maximum load of 125 kg for driver plus any items carried on the hand bike. If you exceed the maximum load, this can lead to damage to the hand bike, loss of control, or to serious injury to the user and other people.

- Exercise caution whenever you use the hand bike. For example, avoid travelling against an obstacle without braking (step, kerb edge) or dropping down gaps.
- Your hand bike has not been designed for transportation in a vehicle. DO NOT sit on the hand bike during any transportation in a vehicle; always use a properly secured vehicle seat!!
- This hand bike should be used exclusively to convey one person at a time. Any other use does not comply with the intended purpose.
- The bike may only be used by one person with a maximum weight of up to 125 kg.
- Use of the bike is only recommended for those who have the right physical and psychological constitution to be able to cope with public traffic.
- As a beginner, you should ride particularly carefully at the start.
- Familiarise yourself with the general traffic laws! In order to achieve a stable ride when travelling straight ahead, wherever possible, you should avoid hasty steering movements when travelling at speed or going round corners.
- When you are riding and braking, always hold the handles firmly with both hands, as otherwise, there is a high risk of an accident through steering error.
- The bike is a mechanical traction device. The speed

must be adjusted to suit the technical capabilities of the bike and the terrain.

- At night and dawn/dusk, always use the lights. When you are travelling long distances, we recommend that you take a spare battery with you.
- Only ride on asphalt paths, or firm, smooth surfaces. When riding over curbs, steps as well as potholes, there is an increased risk of tipping over.
- Adjust your method of driving to your capabilities. Ride slowly and carefully, particularly when riding round corners.
- Do not put your hands in the chain area.
- Your bike should be serviced by your dealer at regular intervals.
- Be careful with fire, in particular with burning cigarettes. Seat and back slings can be set alight.
- With all moving parts, there is always a risk that fingers could become trapped. Please always handle carefully.
- If the hand bike is subject to direct sunlight / low temperatures for long periods, parts of your hand bike could become very hot (>41°C) or very cold (<0°).
- Always make sure that the quick tensioner on the rear wheels is set properly.
- The hand bike should not be used in heavy rain, in snow or on slippery or damaged surfaces. In addition, you should avoid any dangerous areas.
- Make sure that your tyres have sufficient tread!
- Please note that you are subject to any and all traffic laws when riding in public traffic.
- Never use the leg support or the impact protection to lift the hand bike
- Familiarise yourself with the braking distances from various speeds.
- You should only use product combinations which have been approved by Sunrise Medical.

The hand bikes shown and described in this user guide may not correspond in every detail exactly to your own model. However, all instructions are completely relevant, regardless of possible detail differences.

The manufacturer reserves the right to alter without notice any weights, measurements or other technical data shown in this manual. All figures, measurements and capacities shown in this manual are approximate and do not constitute specifications.

Before each use, you should carry out the following safety checks

- Before each journey, check the tyre pressure. If the air pressure is too low, there is an increased risk of tipping over when going round corners, it can lead to increased wear and tear and the rolling resistance will be increased.
- Avoid damage to the Bowden cables for the gearshift and braking units. There is a risk of kinking the cable when you transfer to/from the bike and if you operate the crank incorrectly.
- Check that the footrests are secure.
- Check that all components, in particular all screws, are secure.
- To avoid serious injury, you must secure your feet in the footrests, using the Velcro straps, before starting your journey.
- Always wear a safety helmet.
- You must always ride with a safety flag, as, due to the low height of the bike, you may be less visible to other traffic.

2.0 Warranty

Warranty

THIS DOES NOT AFFECT YOUR LEGAL RIGHTS IN ANY WAY.

Warranty conditions

1) The repair or replacement is carried out by an authorised Sunrise Medical dealer/service agent.

2) To fulfil the guarantee conditions, should servicing need to be carried out on your hand bike under this agreement, contact the designated Sunrise Medical customer service agent immediately, with precise details on the type of difficulty. Should you be using the hand bike outside the area covered by the designated Sunrise Medical customer service agent, the work will be carried out under "guarantee conditions" by another agency as designated by the manufacturer.

3) Should any part of the hand bike require repair or replacement, as a result of a specific manufacturing or material defect, within 24 months (5 years on frame) from the date on which the possession of the product was transferred to the original purchaser, and subject to it remaining within that ownership, the part or parts will be repaired or replaced completely free of charge if returned to the authorised service agent.

Note: This guarantee cannot be transferred.

4) The guarantee also covers all repaired or replaced parts for the remaining period of the guarantee for the hand bike.

5) For spare parts which are fitted after the start of the original guarantee, we give a further 24-months guarantee.

6) Consumable parts are normally excluded from the

guarantee, except in the case that premature wear of the part is the direct result of a manufacturing fault. These items include amongst others upholstery, tyres, inner tubes and similar parts.

7) The above warranty conditions apply to all product parts for models purchased at full retail price.

8) Normally we do not accept responsibility if a repair or replacement of the hand bike is required for one of the following reasons:

a) The product or part has not been maintained or serviced in accordance with the manufacturer's recommendations as shown in the User Instructions and/or the Service Instructions. Accessories have been used which are not specified as original accessories.

b) The hand bike or part having been damaged by neglect, accident or improper use.

c) Alterations to the hand bike or parts, which are not in accordance with the manufacturer's specifications or the carrying out of repairs before informing the customer service agent.

3.0 Handling

Delivery

Your new Shark RS / RT is delivered fully assembled in a cardboard box. In order to avoid any damage during transportation, loose or parts to be fitted, such as the rear wheels are packed separately in the cardboard box.

Unpacking:

- Check the shipping cardboard box for external damage which may have been caused during transportation.
- Remove all packaging material.
- Take all bike parts (including wheels) carefully out of the box and check that they are complete.
- Check all parts for surface damage, scratches, cracks, dents, distortions and other defects.
- Should you find any damage, or if any parts are missing please let us know immediately.

Transporting the Shark RS / RT:

To keep the Shark RS / RT as compact as possible, the rear wheels are removed before delivery and packed separately in the box.

The wheels are mounted to the rear axle by simply pushing them on or by screwing them on, depending on your choice of axles.

⚠ CAUTION!

Before each use, you must check that the screw and plug connections are secure!

4.0 Options

Brake

Brakes

The bike has two brakes which work independently from one another.

Parking brake (Fig. 4.1 + 4.2)

The parking brakes, mounted on the fork serve as wheel locks. They can also be applied with the aid of a locking lever. The locking lever must be locked into place for these to work properly. The bike should not roll when the parking brakes are applied. This is the only way to guarantee safe transfer to and from the bike.

Main brakes (Fig. 4.3)

The brakes allow you to safely and comfortably brake your hand bike.

⚠ CAUTION

If the main brakes should fail, then the parking brakes may be used once for emergency braking. After that, the main brakes must be repaired.

Fig. 4.1

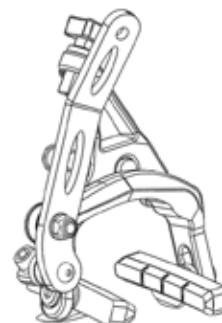
Fig. 4.2


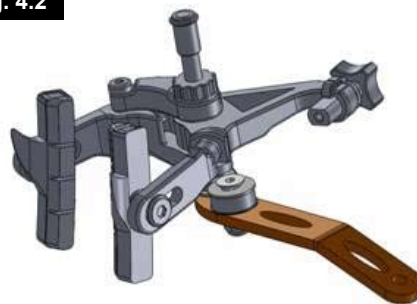
Fig. 4.3

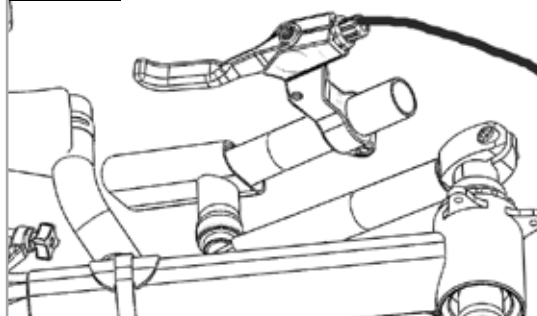⚠ CAUTION!

Braking power will decrease with:
Worn tyre tread, air pressure which is too low, soiled and wet tyres.
But the ground surface can also play a significant role in delaying the braking effect. The braking effect will be significantly reduced if the streets and paths are wet and dirty, as well as on sandy or stony ground. Dirty and wet rims and brake pads also reduce the braking effect when using rim brakes.
For this reason, you should always adapt your method of riding to the environmental conditions in order to avoid serious accidents.
Always check the efficiency of the brakes before starting your journey and ensure that there is sufficient brake function. The brakes should be checked regularly by your dealer and adjusted if required.

Back pedal brake (Fig. 4.4 - 4.6)

The back pedal brake option allows you to brake by pedalling backwards, when used in combination with chain gearshift (Fig. 4.5). This option is required particularly for users with restricted or no finger function.
 The braking system can only be uncoupled for manoeuvring (Fig. 4.6).
 When riding or moving forward, use the rocker arm to put the system into an operational position.



⚠ CAUTION!

Riding with the back pedalling brake deactivated is not permitted and can lead to an increased risk of accidents.

Footrest

Adjusting the footrest (Fig. 4.7 - 4.9)

The footrests can be adjusted to suit the leg length.
 . The adjustment can be made after releasing the screws (1). When you adjusting the length, you must make sure that the legs are slightly angled and that they touch the bar at the front. With the bike at a standstill, you should check that when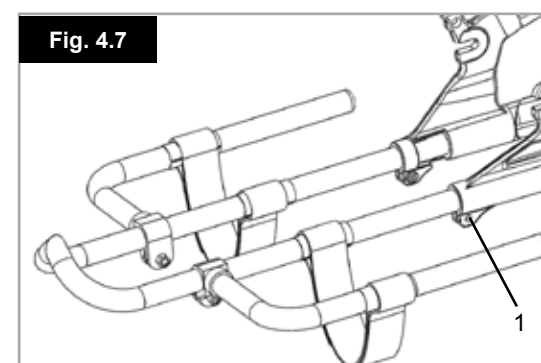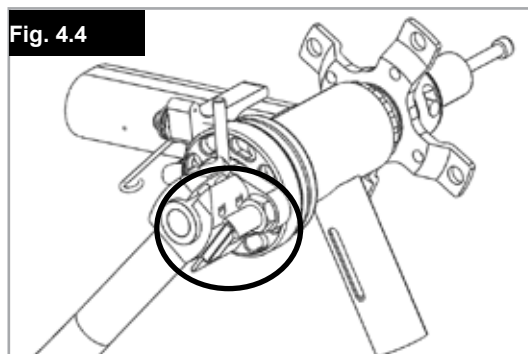

turning at the maximum angle, the increased knee bend does not lead to contact with the drive crank. There must be a gap of at least 5 cm between the heel and the floor. You will need this leeway when travelling round corners. When finished, make sure that all screws have been properly tightened (see the page on torque).

The footrest tubes for longer legs are always delivered at the maximum length. The customer can then shorten them to the required length.

After shortening the tube, you must always ensure that the footrest tube is pushed into the fork far enough so that both fixing screws touch the footrest tube.

⚠ CAUTION!

Never use the leg support to lift the hand bike. Before use, you must secure your feet into the footrests. If your feet accidentally come out when you are riding, this can lead to serious injuries.

RT gearshift

Chain gearshift/hub gearshift (Fig. 4.10 - 4.11)

Gear shift is carried out using the Grip Shift switch, which can be mounted either on the frame, the fork or the handle.




Shifting the upper chain rings on the crank is carried out using a lever which is mounted on the crank housing.

The chain gearshift enables you to easily keep moving, regardless of other road conditions.

A description of the gear change unit is included as part of the delivery. If the gearshift suddenly does not operate smoothly, please contact your dealer!

RS gearshift (Fig. 4.12)

This is carried out using the trigger switch on the right handle or the Dual Control switch on the right-hand side of the fork.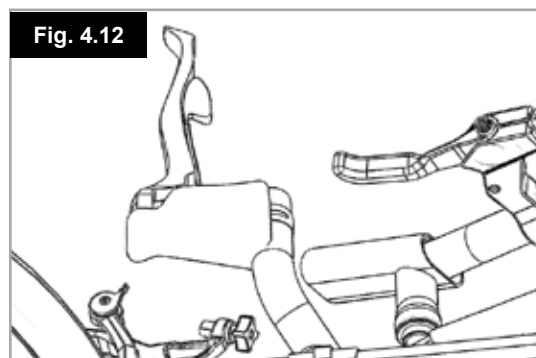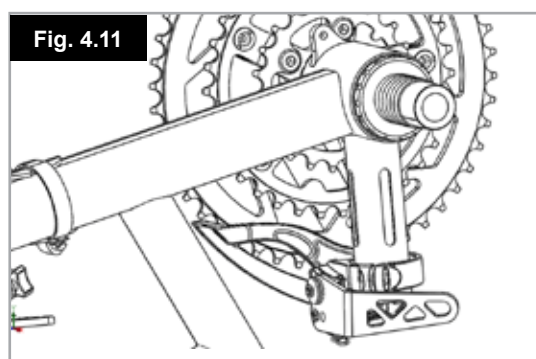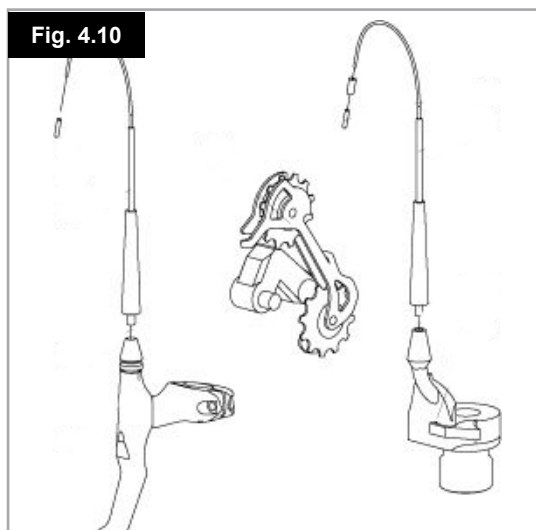

Backrest adjustment




Backrest adjustment (Fig. 4.13 - 4.14)

During the seating test ensure that the cranks turn freely above the thighs, even when going round a corner. (Test backwards in freewheel, adjust seat cushion thickness if necessary). Set the gap from the crank so that, when turning the crank, your arms stretch to a comfortable distance and can bend easily. The very best drive characteristics are achieved if you use the adjustment options for the backrest and the crank height, in order to establish the optimum position for you.

Adjusting the backrest is carried out lengthways by releasing and moving the fixing clamp. Adjusting the backrest angle is carried out by adjusting the height of the central tube behind the backrest.

⚠ CAUTION!

After making the adjustment, re-tighten all screws. (see the page on torque)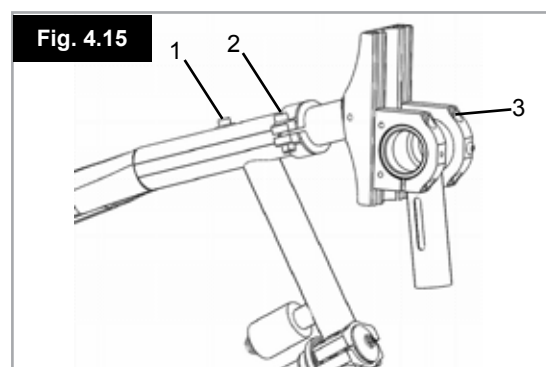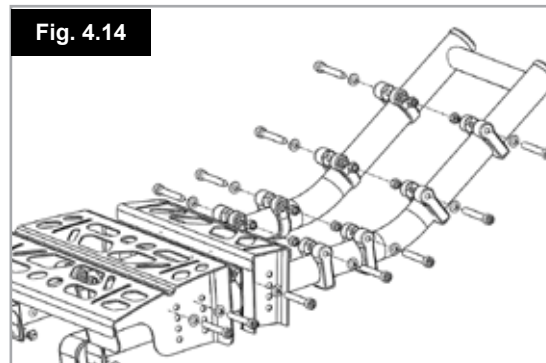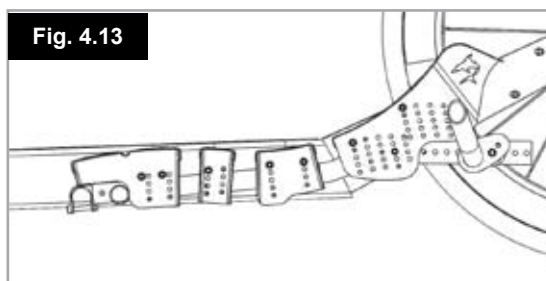

Height and depth adjustable fork

Diagonal adjustment (Fig. 4.15)

Undo the socket head screws (1) on the fork tube and the hex nuts (2) on the clamp. Push the crank bracket forwards or backwards. When the crank is in the required position, the chain tension will not be correct, so to correct the tension, chain links **MUST** be added or removed. This procedure should be carried out by a qualified technical engineer. Finally, re-tighten the socket head screws which fix the crank bracket to the fork.

Vertical adjustment

Undo the socket head screws (3) which fix the crank bracket to the fork stem, but do not remove them.

Move the hand crank unit up or down in the fork collar, until you have achieved the optimum position for you.

Tighten the socket head screws (3) to fix the hand crank in the required position.

Re-tension the chain if necessary.

Rear axle

Rear axle (Fig. 4.16 - 4.17)

There are 3 different axles in the range, which can each be screwed to the frame using the axle clamp.

Swap / change the axle:

Undo the 3 fixing screws on the left and right axle clamp. Remove the axle and exchange it.

To complete the operation, place the half shell of the axle clamp back onto the axle and fix it with the three fixing screws.

Tighten the 3 fixing screws alternately until all 3 screws are fixed to a torque of 5 Nm.

Note: When changing from aluminium to carbon tubes, the eccentric sleeve must also be changed.

Fig. 4.16


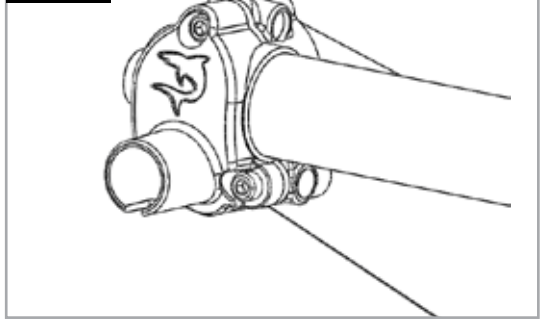
Fig. 4.17

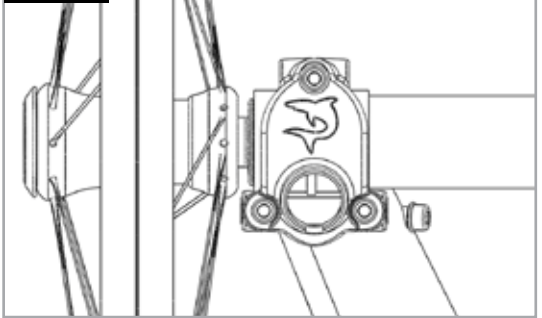5.0 Maintenance and care (checks to be carried out before use)

When	What	Comment
Every 4 weeks (depending on frequency of use)	Grease the cables where they leave the switches and the brake levers. Check all screw connections are secure! Oil the drive chain!	Carry out yourself or with an attendant. Before oiling, please remove all old oil remnants from the components. Make sure that the surplus oil does not get the surrounding area (e.g. your clothes) dirty. Carry out yourself or with an attendant. Spray lightly with Teflon spray Be careful not to get it on your clothes!
Every 2-3 months (depending on frequency of use)	Check: • Cleanliness • General condition	Carry out yourself or with an attendant. If there are any cracks or general wear and tear, then change the rubbers or visit an authorised service centre for a refurbishment.

Service:

If you have any questions or require any help, please contact your SOPUR dealer, who will be pleased to advise you with regards to servicing and repairs.

Depending on how much you have used your Shark RS / RT, we recommend that you take it to your dealer at least every 6 months, to have your Shark RS / RT inspected by trained personnel.

6.0 Tyre maintenance

Tyres

The Shark RS / RT is equipped with special 26" tyres on the rear wheels and a 26" drive wheel. Make sure that they are always at the recommended tyre pressure, as this will significantly affect the drive characteristics of your Shark RS / RT. If the tyre pressure is too low, movement and manoeuvrability is made more difficult due to the increased rolling resistance and this also increases the amount of effort required.

The optimum tyre pressure is printed on each tyre.

Fitting and repairing tyres

Before fitting a new inner tube:

- Inspect the base of the rim and the tyre inner wall for foreign bodies and if necessary clean thoroughly.
- Check the condition of the rim tape, particularly at the valve hole. Only well fitting, strong rim tape offers long-term protection against inner tube damage through burrs and sharp spoke heads.

Inflation:

Check all round on both sides, to make sure that the inner tube is not trapped between the tyre bead and the rim. Push the valve back gently and pull it back out, so that the inner tube achieves an optimum position around the valve.

Tube tyres

As an option you can have tube tyres with full carbon rims.

With these, please note:

Tube tyres must be glued to the rim by a specialist.

Should you have a puncture, you must not continue to ride (the immediate consequences are an increased risk of accident and the rim will be destroyed).

CAUTION!

After fitting, check the tyre pressure.

It is essential that you maintain regulation tyre pressure and keep the tyres in perfect condition, in order to ensure your personal safety and keep your Shark RS / RT running properly.

7.0.0 Disposal / Recycling of Materials

NOTE: If the hand bike has been made available to you as part of a charity or medical loans scheme, then it may not belong to you. If it is no longer required, then follow the instructions to return it as given by the organisation that made the hand bike available to you.

In the following section, there is a description of the materials used on the hand bike, in view of the disposal or recycling of the hand bike and its packaging.

There may also be special local regulations in force with regard to disposal or recycling, these must be taken into account when disposing of your hand bike. (This can include the cleaning or decontamination of the hand bike prior to disposal).

Aluminium: Castor Forks, wheels, footrest,

Steel: Fixing points,

Carbon: Rear axle

Plastic: Handles, tube stoppers, armpads and tyres

Packaging: Made of soft polyethylene, cardboard

Upholstery: Woven polyester with PVC coatings and expanded combustion modified foam.

Disposal or recycling should be done through a licensed agent or authorised place of disposal. Alternatively your hand bike may be returned to your dealer for disposal.


8.0 Nameplate

Nameplate

The nameplate is located on either the rear part of the frame or on the frame underneath the seat surface, as well as on a label in the owner's manual. The nameplate indicates the exact model designation and other technical specifications. Please provide the following pieces of information whenever you have to order replacement parts or to file a claim:

As we are committed to continually improving the construction of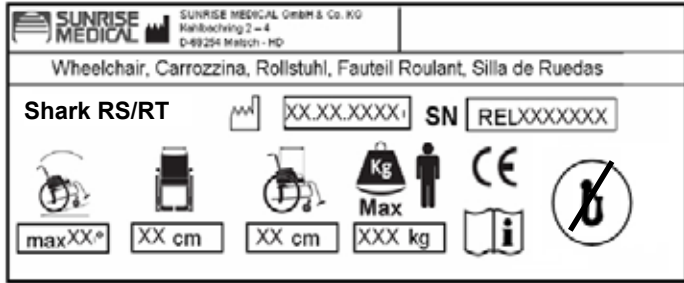

Shark RS/RT XXXXX-XXX	Product name/SKU number
	Maximal safe slope with anti-tips depends on hand bike settings, posture and physical capabilities of the user
	Seat Width
	Depth (maximum).
	Maximum load.
	CE marking.
	Directions for use
	not crash tested
	Manufacturing date.
	Serial number
SN RELXXXXXXXX	

our hand bikes and hand bikes, product specifications may vary slightly from the examples illustrated. All weight/dimensions and performance data is approximate and is provided solely for guidance.

All products from Sunrise Medical comply with the 93/42/EEC directive for medical products


CE All hand bikes must be used in accordance with the manufacturer's guidelines.

Sunrise Medical GmbH & Co.KG
 Kahlbachring 2-4
 69254 Malsch/Heidelberg
 Germany
 Tel.: +49 (0) 7253/980-400
 Fax: +49 (0) 7253/980-111
 www.sunrisemedical.de

9.0 Torque

Steering head bearings:	Handtight
Castor fork clamps:	7 Nm
Screw axle on drive wheel:	10 Nm
Footrest clamp screws:	7 Nm
Crank clamp screws:	7 Nm
Front backrest clamp screws:	7 Nm
Rear backrest clamp screws:	7 Nm

Torque for M6 screws is 7 Nm, unless otherwise specified.

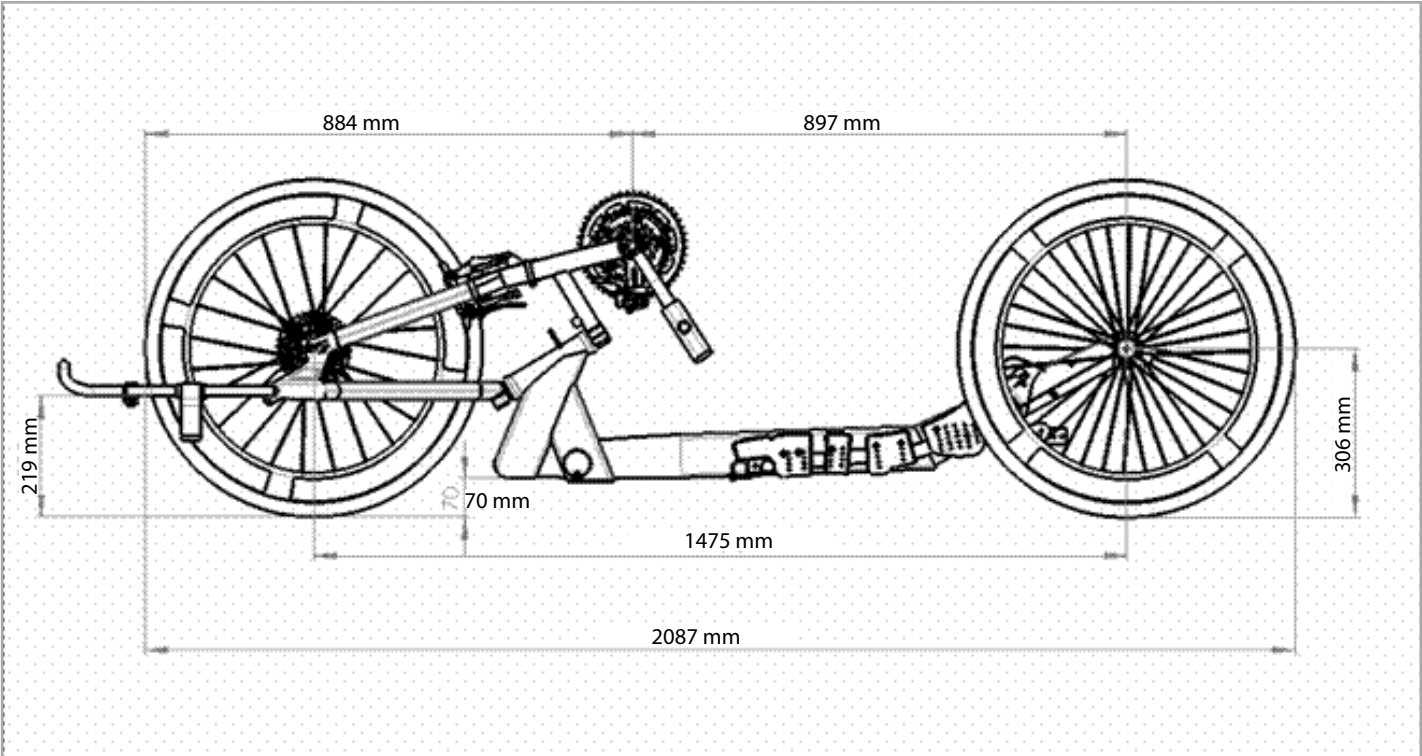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Sunrise Medical AB
Sunrise Medical B.V.
G2 A/S
Sunrise Medical Pty. Ltd.